Latexfreie Kondome:

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Latex-Free Condoms:

Medical product

Please read the instructions for use carefully

Protect condoms from direct sunlight and heat

Do not use if the packaging is damaged

Protect from moisture

Condoms are only intended for single use

Latex-free condoms are made from polyisoprene. They are free of natural rubber latex and are suitable for people with a known or suspected latex allergy.

If you experience any discomfort or irritation, stop using the condom and ask your doctor for advice.

Each condom is individually electronically tested. Correct use of condoms reduces the risk of unwanted pregnancy and infection with sexually transmitted diseases such as HIV/AIDS. Condoms offer effective protection against pregnancy and sexually transmitted diseases. However, as with all contraceptives, 100% protection cannot be guaranteed. If you have unprotected sex or if the condom tears or bursts during use, please seek medical attention as soon as possible, but within 72 hours. Please report any serious incidents relating to the product to the manufacturer and your competent national authority.

Dangers of reusing condoms: cross-infection, leakage due to cleaning, loss of lubricity and an increased risk of tearing. All of these factors increase the risk of pregnancy or transmission of a sexually transmitted disease.

In some highly sensitive people, ingredients in these condoms can cause allergic reactions. If irritation or burning occurs, stop using this condom and consult a doctor.

Do not use oil-based lubricants such as massage oil, body lotions, baby oil, butter, margarine, petroleum jelly, etc., as polyisoprene condoms can be damaged by oil-containing substances. If you use lubricants, they must be labeled as suitable for use with condoms. Before using lubricants, please read the instructions for use on the packaging. Do not apply lubricants directly to the penis or the inside of the condom, as this increases the risk of the condom slipping off or tearing.

Non-vaginal use of condoms, i.e. oral sex (as teeth can damage the condom) and anal sex (as there may be a lack of natural lubrication), increases the risk of slipping and damage. When using vaginal suppositories, please read the respective instructions for use. If prescription or non-prescription medications (ointments, gels) are used in the area of the penis and vagina, please ask your doctor or pharmacist.