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Safety information:

Note on the Medical Devices Act (MPG)

Medical devices are products with a medical purpose that the manufacturer intends for use in humans. Unlike medicinal products, which have pharmacological, immunological, or metabolic effects, the intended main effect of medical devices is achieved primarily through physical means, for example.

As far as affected/applicable, our products are medical devices and are subject to the provisions and regulations of the Medical Devices Act.

TENS (Transcutaneous Electrical Nerve Stimulation) is considered a method with minimal risks and few side effects. When used properly, the method is considered to be relatively free of side effects.

TENS/EMS devices should not be used, or should only be used under the advice of your doctor or therapist:

for pacemaker or defibrillator wearers

for pregnant women

for epilepsy

- Read the electrical stimulation device's instructions before use

- Do not use while driving a vehicle

TENS/EMS electrodes should not be placed directly:

on the head and carotid arteries

on the larynx

over metal implants

on wounds and malignant skin/tissue changes

not directly over the heart

In rare cases, slight skin irritation may occur. This may be related to the chosen intensity of the treatment or to an allergic skin reaction to the electrodes used or the electrode coating.