Anwendung:

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Application:

Apply the desired amount with your fingers to your intimate area or to your favorite toy. When using condoms, apply the lubricant to the outside of the unrolled condom.

Cleaning:

AQUAglide can be completely rinsed off with water. Dried residues on your favorite bed linen can be removed without leaving stains using commercially available cleaning products.

The AQUAglide Original can also be completely removed from your favorite toy. For this we recommend a cleaning and disinfectant

Warnings:

AQUAglide is not a contraceptive, does not contain any spermicidal agents and does not offer any protection against infections. Gynecologists recommend using lubricants sparingly or avoiding them if you want to have children, as they could make conception more difficult.

Contraindications:

If you have known allergies or are sensitive to certain ingredients, you can find them under "Ingredients (INCI)". If you experience any intolerances, please wash off the lubricant thoroughly and contact your general practitioner or gynecologist if necessary.

If you have any further questions, please do not hesitate to contact our team. Your gynecologist, a pharmacist or a sexual counselling center will also be able to help you.